Abb./Fig. 1

REF 300101ST
UDI 764016111004P

DE KATHETERVENTIL
FR VALVE DE CATHÉTER
IT VALVOLA CATETERE
EN CATHETER VALVE
NL KATHETER VENTIEL
ES VÁLVULA DE CATÉTER
EL βαλβίδα καθετήρα

KV 200 EH

STERILE EO
CE 0297

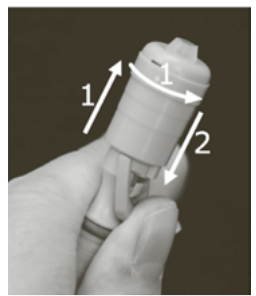
Abb./Fig. 2

Max Stäubli AG
Spätzstrasse 14, CH-8810 Horgen
Tel. +41 (0)44 728 80 40
Fax +41 (0)44 728 80 41
info@maxstaeubli.ch, www.maxstaeubli.ch

EC REP MED-RAS GmbH
Eichenallee 8H, D-21521 Wohltorf, Germany
Tel.: +49 (0)4104 99 44 44-0
Fax: +49 (0)4104 99 44 44-9
info@medras.de, www.medras.de

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EN  These instructions for use must be read carefully before using the catheter valve. Your attending healthcare professional will instruct you on how to use the catheter valve.

1. Product description/Intended use

The catheter valve consists of a polyamide, spring steel and silicone connector. It is used to seal fitted urinary catheters. It is connected to the catheter tube, allowing patients to empty their bladder (micturition) in a self-sufficient, controlled manner. It is used outside the human body.

A catheter valve is intended for single use on a single patient during a single treatment. It is not reusable. A "single treatment" is defined as the period during which a urinary catheter (Cystofix or similar) forms a closed system with the valve in/on the patient's body. Changing the urinary catheter is defined as another treatment. Removing the catheter valve from the urinary catheter is also considered another treatment and requires the use of a new catheter valve in sterile packaging. Max Stäubli AG recommends a maximum duration of use of three (3) months.

2. Contraindications

Catheter valves are not suitable for

- patients with ureteral reflux
- people with severe cognitive impairment
- people with severe obesity
- people with motor impairment
- in cases of tentorial or suspected formation of urinary stones (bladder or kidney stones) → risk of blockage (urinary retention symptoms) by a spontaneously released urinary stone
- uncontrolled detrusor hyperactivity
- renal insufficiency
- small bladder capacity
- the catheter valve must not be allowed to lie on open wounds
- use for too long a period can lead to malfunctions and, as a result, to symptoms of urinary retention.

3. Target group and intended users

Catheter valves are equally suitable for men and women who have been fitted with a short- or long-term urethral or suprapubic catheter. The valve is fitted to the patient by a medical professional in a clinical environment after a urinary catheter has been fitted.

The valve is operated (opened, closed) by the patient. This is done in the "normal" environment in which the patient is located or able to stay given the state of the patient's health.

4. Warnings

- Various metabolic waste products normally excreted in the urine, such as kidney stones, bladder stones or urate crystals, can lead to malfunctions. The catheter valve must always be replaced when the catheter is changed. If the patient is known to have calculosis, the valve should be checked regularly to ensure that urine can flow through it, or replaced if necessary.
- In the event of suspicion/symptoms of urinary retention (absence of or decrease in flow through the open valve), the doctor must be consulted.
- If the catheter valve or coupling piece comes into contact with injured skin, this can lead to inflammations.
- When the catheter valve is closed under unfavourable conditions, skin can become caught in the valve.
- In the event of mechanical damage to the catheter valve or parts of it (e.g. the trigger), there is a risk of injury to the skin.
- If the catheter valve is deliberately tampered with, e.g. by removing the trigger, it will no longer be possible to operate it one-handed.
- In the event of damage to the catheter valve, consult the doctor and arrange a replacement of the valve.
- Lying on the catheter valve for too long can lead to decubitus (bed sores).
- When using a coupling piece, the connected urine bag must be placed lower than the user and have a backflow prevention device to reduce the risk of infection.
- Always wash your hands with soap before and after handling the catheter, catheter valve or urine bag.

5. Use/Operation

Before opening the sterile packaging, check it for integrity. The indicator for sterility must be orange. When unpacking, follow the opening direction indicated on the packaging and ensure that the packaging is torn open no more than 2/3. In addition, that during the insertion of the catheter valve it is not soiled and not damaged. Products which are found to be damaged and deviating must be disposed of. See chapter 10.

Fig. 1

Fitting the catheter valve: Ensure that the end of the tube and the funnel are dry.

Insert the conical nipple on the catheter valve into the cone on the catheter tube until the green ring is covered. Check the functioning of the system: tight fit of the valve in the tube, urine drainage and tightness after closure.

Fig. 2

1) Two-hand operation: Release urine (open)

Hold the catheter valve by the conical nipple. Pull the sliding sleeve forward and make a slight clockwise turn. Valve remains open.

Close the valve: Turn the sleeve slightly anticlockwise. The valve closes again.

2) One-hand operation: Release urine (open)

Hold the valve as shown in figure 2 and push the orange bracket backwards (direction of arrow) with your thumb.

Close the valve: As soon as all urine has drained, the orange bracket can be released and the valve closes automatically.

Connecting the coupling KP 200 to the catheter valve: see instructions for use coupling piece.

6. Cleaning/personal hygiene

Cleaning is carried out while connected. Clean the catheter valve and the genital area at least once a day with water and pH neutral soap. Replace the washcloth daily or use single-use washcloths.

Remove incrustations: these can be sources of infection.

Showering and bathing are permitted. When doing so, clean the catheter valve under running water.

7. Duration of use

The duration of use corresponds to the catheter replacement cycles/intervals. However, a valve must always be replaced:

- when the catheter is replaced (doctor's visit)
- as soon as the drainage speed decreases (lower flow)
- approximately three months after fitting to the catheter

Improper reuse (misuse) of the catheter valve can lead to inflammation of the urinary tract.

8. Possible faults and troubleshooting

The following faults may occur and are known potential residual risks:

Fault/Malfunction	Effect	Possible cause	Workaround/action
Valve not impermeable	Urine draining uncontrollably	Spring tension has decreased because the valve has been used/worn for too long	Replace the catheter valve.
System not impermeable	Urine draining uncontrollably	The urinary catheter tube does not fit the diameter of the catheter valve: does not form a seal	Consult a doctor to fix the cause of the problem.
No seal between the coupling piece and the valve	Urine draining uncontrollably	The coupling piece has not been correctly fitted to the catheter valve	Check that the coupling has correctly clicked into place on the valve, see also chapter 5 of the instructions for use for the coupling piece. If necessary, remove the coupling piece and reattach.
System impermeable	Urine cannot drain	Catheter valve does not open	Remove the catheter valve to allow the urine to flow, with assistance from others if necessary. If this is not possible, seek medical treatment immediately. Reseal the catheter with a new valve.
		Possible residue in the system: tube or valve blocked	Remove the catheter valve to allow the urine to flow, with assistance from others if necessary. Then consult a doctor to determine the cause of the blockage and take appropriate action.
Catheter valve damaged	Trigger has broken off	Kinks in the tubes	Straighten the kinks; if this does not help, take action as above.
		Risk of injury, more difficult to handle, one-handed operation no longer possible	Valve can still be opened by pulling the sliding sleeve forward: Two-hand operation → hold the catheter tube firmly by the cone of the valve with the second hand! The valve must be replaced.
	Trigger is hanging loose	More difficult to handle, one-handed operation no longer possible	

9. Product Complaints

Should a malfunction of a catheter valve ever lead to a “serious incident” the person concerned must inform Max Stäubli AG and the authorities in his or her country without delay. For every reporting, please indicate the name and article number **[REF]** of the component, the lot number **[LOT]**, your name and address, and the nature of the complaint.

10. Disposal

The devices can be disposed of via the established systems available to users (household waste).

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