Abb./Fig. 1

Abb./Fig. 3

REF 300200ST
UDI 764016111004P

stäubli
SWITZERLAND

DE KUPPLUNGSSTÜCK
FR ACCOUPLEMENT
IT RACCORDO
EN COUPLING
NL KOPPELING STUK
ES ACOPLAMIENTO
EL Εξάρτημα

KP 200

CE 0297

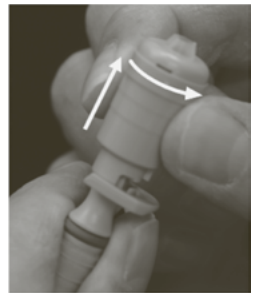
Abb./Fig. 2

Max Stäubli AG
Spätzstrasse 14, CH-8810 Horgen
Tel. +41 (0)44 728 80 40
Fax +41 (0)44 728 80 41
info@maxstaeubli.ch, www.maxstaeubli.ch

EC REP

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EN  The instructions for your catheter valve also apply. Please note the information listed there.

Your attending healthcare professional will instruct you on how to use the catheter valve and the coupling.

1. Product description/Intended use

The coupling piece is used for connecting a urine bag. This allows the urine to drain without obstruction and the patient to sleep through the night.

2. Contraindications

None known.

3. Target group and intended users

See instruction catheter valves.

4. Warnings

See instruction catheter valves.

5. Use/Operation

Before opening the sterile packaging, check it for integrity. The indicator for sterility must be orange. When unpacking, follow the opening direction indicated on the packaging and ensure that the packaging is torn open no more than 2/3. In addition, that during the insertion of the coupling piece it is not soiled and not damaged. Products which are found to be damaged and deviating must be disposed of. See chapter 10.

All operations must be carried out with the catheter valve closed.

Fig. 1 Connect the KP 200 coupling piece to the end of the urine bag tube.

Make sure the diameter of the urine bag tube fits the coupling and forms a seal: Tight fit of the coupling in the tube

Connecting the coupling piece to the catheter valve:

The KP 200 coupling piece and the catheter valve (KV 100 or KV 200 EH) must be exactly aligned with each other and pushed together until a click is heard.

Fig. 2 **Opening the catheter valve:**

Hold the valve at the tapered end with one hand and pull the sleeve all the way forward with the other hand. Rotate while pulling so that the valve remains open.

The coupling piece must not be twisted any further after connection. If this happens, the catheter valve will seal again and the flow of urine will no longer be ensured.

Risk of urine retention symptoms.

Urine bag must be attached below the catheter valve to ensure drainage. The use of a urine bag with a non-return valve is recommended.

Fig. 3 **Removing the coupling piece:**

Press the two tabs together gently, tilt the coupling piece slightly and pull it out.

6. Cleaning/personal hygiene

First close the catheter valve, then remove the coupling piece from the catheter valve before pulling it out of the urine bag. Now rinse the coupling piece well with hot water and clean with pH-neutral soap.

7. Duration of use

The duration of use corresponds to the cycles/intervals of catheter replacement. A coupling piece should always be replaced when a new catheter or a new valve is used.

8. Possible faults and troubleshooting

The following faults may occur and are known potential residual risks:

Fault/Malfunction	Effect	Possible cause	Workaround/action
No seal between the coupling piece and the valve	Urine draining uncontrollably	The coupling piece has not been correctly fitted to the catheter valve	Check that the coupling piece has clicked correctly into place on the valve as described in section 5 of the instructions for use, and if necessary, remove the coupling piece and reattach.

Other problems related to the catheter valve are described in the instructions for the catheter valve.

9. Product Complaints

Should a malfunction of a catheter valve ever lead to a "serious incident" the person concerned must inform Max Stäubli AG and the authorities in his or her country without delay. For every reporting, please indicate the name and article number [REF] of the component, the lot number [LOT], your name and address, and the nature of the complaint.

10. Disposal

The devices can be disposed of via the established systems available to users (household waste).

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